**PRODUCT DESCRIPTION**  
 BeautiSealant is light-cured pit and fissure sealant system, consisting of 'Paste' and 'Primer'. Primer modifies enamel surface due to its adhesive monomer to bond Paste to natural enamel.

**INDICATIONS FOR USE**  
 Pit and fissure sealant (Primer is intended to bond Paste to natural enamel.)

**PRECAUTIONS AND WARNINGS**

- Do not use this product on patients with known allergies to this material, methacrylate monomer and/or acetone.
- Operators with known allergies to methacrylate monomer and/or acetone must not use this product.
- If any inflammation or other allergic reactions occur on either patient or operator, immediately discontinue use and seek medical advice.
- This product should not come in contact with eyes, skin or clothing. In case of accidental contact with skin or clothing, immediately blot with alcohol moistened cotton ball.
- Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers may penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water. In case of contact with eyes, immediately flush eyes with plenty of water and seek medical advice.
- If the primer comes in contact with oral soft tissue, the tissue may turn whitish, however, it should resolve in 2 or 3 days. Avoid touching or brushing the area.
- The brushes, dishes (V-DISH) and delivery tips (needle tips) are for single use only.
- Use this product in a well ventilated area (several appropriate ventilations per hour are recommended).
- Avoid any source of ignition while using Primer.
- Use all the dispensed material promptly on one patient to avoid cross infection.
- Do not use this product in conjunction with other pit and fissure sealants and/or adhesives.
- Use saliva ejector during procedures to eliminate water or saliva in patient's mouth.
- Follow the instructions for use of each dental material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package and container.
- (Example YYYY-MM-DD → Year-Month-Date of the expiration date)
- This product is intended for use by dental professionals only.

**DIRECTIONS FOR USE**

- Cleaning of tooth surface  
 Clean the tooth surface following the standard method.
- Isolation
- Isolate with rubber dam etc following the standard method.
- Preparation
  - When storing these materials in a refrigerator, allow them to reach room temperature before use.
  - Attach a needle tip following the procedures below:
    - Remove the shield cap by turning it slowly (Fig. A).
    - Attach a needle tip by turning it in clockwise until it is securely attached (Fig. B).
- Pretreatment of tooth surface  
 Dispense adequate amount of Primer on a V-DISH. Apply on the enamel surface of the pit and fissure using a brush. Leave undisturbed for 5 sec. Gently air blow for about 3 sec and then dry with stronger stream of air until a thin and uniform bonding layer is obtained.
- Primer must be closed immediately after dispensing as it contains volatile components.
- Filling of the paste  
 Apply necessary amount of the paste directly to the pit and fissure or it may also be extruded onto a paper pad to be applied to the pit and fissure with a suitable instrument.
  - The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.
- Light-curing  
 Light-cure using a dental light-curing unit.

**Light-curing unit**

Light-curing unit	Halogen	LED
Light-curing time	20 sec.	10 sec.
Wavelength	400 - 500 nm	440 - 490 nm
Light intensity	≥ 500 mW/cm <sup>2</sup>	≥ 1000 mW/cm <sup>2</sup>

➢ Use a dental light-curing unit having specifications equivalent to those described above. When using the one with higher performance scale other than those recommendations, follow the instructions of each unit to light-cure BeautiSealant properly.

➢ Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light. Operator must also protect patient's eyes from the curing light.

➢ Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.

7. Removal of uncured material  
 Remove uncured material on the surface with an alcohol moistened cotton ball etc.

**COMPOSITION**  
 Paste: Fluoroboroaluminosilicate glass, UDMA, TEGDMA, Micro fumed silica and others  
 Primer: Acetone, Distilled water, Carboxylic acid monomer, Phosphonic acid monomer and others

**STORAGE**

- Store Primer in refrigerator (1-10 °C/34-50°F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Store Paste at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Store Paste in either way of the following.

- Discard the used needle tip and attach new one. Insert the needle into the small hole on the head of the shield cap (Fig. C).
- Discard the used needle tip and replace the shield cap securely (Fig. D).

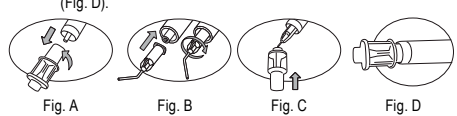
Fig. A Fig. B Fig. C Fig. D

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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5. Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers may penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water. In case of contact with eyes, immediately flush eyes with plenty of water and seek medical advice.

6. If the primer comes in contact with oral soft tissue, the tissue may turn whitish, however, it should resolve in 2 or 3 days. Avoid touching or brushing the area.

7. The brushes, dishes (V-DISH) and delivery tips (needle tips) are for single use only.

8. Use this product in a well ventilated area (several appropriate ventilations per hour are recommended).

9. Avoid any source of ignition while using Primer.

10. Use all the dispensed material promptly on one patient to avoid cross infection.

11. Do not use this product in conjunction with other pit and fissure sealants and/or adhesives.

12. Use saliva ejector during procedures to eliminate water or saliva in patient's mouth.

13. Follow the instructions for use of each dental material, instrument or equipment to be used in conjunction with this product.

14. Do not use this product for any purposes other than specifically outlined in these instructions for use.

15. Use this product within the expiration date indicated on the package and container.

16. (Example YYYY-MM-DD → Year-Month-Date of the expiration date)

17. This product is intended for use by dental professionals only.

**DIRECTIONS FOR USE**

- Cleaning of tooth surface  
 Clean the tooth surface following the standard method.
- Isolation
- Isolate with rubber dam etc following the standard method.
- Preparation
  - When storing these materials in a refrigerator, allow them to reach room temperature before use.
  - Attach a needle tip following the procedures below:
    - Remove the shield cap by turning it slowly (Fig. A).
    - Attach a needle tip by turning it in clockwise until it is securely attached (Fig. B).
- Pretreatment of tooth surface  
 Dispense adequate amount of Primer on a V-DISH. Apply on the enamel surface of the pit and fissure using a brush. Leave undisturbed for 5 sec. Gently air blow for about 3 sec and then dry with stronger stream of air until a thin and uniform bonding layer is obtained.
- Primer must be closed immediately after dispensing as it contains volatile components.
- Filling of the paste  
 Apply necessary amount of the paste directly to the pit and fissure or it may also be extruded onto a paper pad to be applied to the pit and fissure with a suitable instrument.
  - The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.
- Light-curing  
 Light-cure using a dental light-curing unit.

**Light-curing unit**

Light-curing unit	Halogen	LED
Light-curing time	20 sec.	10 sec.
Wavelength	400 - 500 nm	440 - 490 nm
Light intensity	≥ 500 mW/cm <sup>2</sup>	≥ 1000 mW/cm <sup>2</sup>

➢ Use a dental light-curing unit having specifications equivalent to those described above. When using the one with higher performance scale other than those recommendations, follow the instructions of each unit to light-cure BeautiSealant properly.

➢ Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light. Operator must also protect patient's eyes from the curing light.

➢ Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.

7. Removal of uncured material  
 Remove uncured material on the surface with an alcohol moistened cotton ball etc.

**COMPOSITION**  
 Paste: Fluoroboroaluminosilicate glass, UDMA, TEGDMA, Micro fumed silica and others  
 Primer: Acetone, Distilled water, Carboxylic acid monomer, Phosphonic acid monomer and others

**STORAGE**

- Store Primer in refrigerator (1-10 °C/34-50°F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Store Paste at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Store Paste in either way of the following.

- Discard the used needle tip and attach new one. Insert the needle into the small hole on the head of the shield cap (Fig. C).
- Discard the used needle tip and replace the shield cap securely (Fig. D).

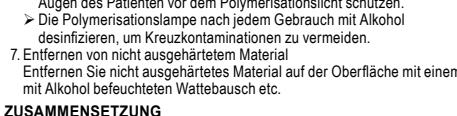
Fig. A Fig. B Fig. C Fig. D

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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