

Dental

Your Source for Product Evaluations
and Information

PRODUCT SHOPPER®

www.dentalproductshopper.com

SHOFU DENTAL CORPORATION

BEAUTIFIL II

Beautiful II has employed
unique chemistry to establish
more than 13 years of clinical
success.



SHOFU DENTAL CORPORATION

Beautiful II

Beautiful II has employed unique chemistry to establish more than 13 years of clinical success.



Beautiful II is designed for direct restorations of all classes that require optimum esthetics and biocompatibility. This fluoride-releasing, highly esthetic, universal nanohybrid composite was the focus of a 13-year clinical study from the University of Florida set to be released this year.

This study found that “25 (61%) of the 41 restorations observed had either excellent or satisfactory ratings after 13 years of placement.” The researchers continued, “These overall positive results may be related to the unique Giomer filler particles.”

In a Glass by Itself

The science behind Beautiful II helps set it apart from

the competition. Beautiful II is a Giomer product, so it incorporates Shofu’s surface pre-reacted glass (S-PRG) technology. S-PRG fillers are obtained by reacting acid-reactive, fluoride-containing glass with polyacids in the presence of water. Beautiful II’s S-PRG fillers release fluoride and recharge, and the fluoride is responsible for reducing tooth mineral solubility, encouraging remineralization of tooth enamel, and decreasing the acid production of cariogenic bacteria.

In addition to the S-PRG fillers, discrete nano fillers (10 to 20 nm) are included in the filler structure to obtain a filler load of 83.3% by weight for fast and easy polishing with an outstanding surface luster that remains over time.

"Excellent composite...Great finish for anterior and posterior teeth."



—VW from Miami, 4/29/13

Online user review posted on
DentalProductShopper.com

This proprietary process, according to Shofu, combines the strength and esthetics of a conventional composite with the ability to release and recharge fluoride and 5 other ions that are known for their bioactive properties that can inhibit bacterial colonization and minimize plaque adhesion.


Feature-Rich

Shofu's Beautifil II offers excellent shade matching through its chameleon effect, with optical characteristics designed to imitate the natural esthetics of both dentin and enamel combined. It is also



VIDEO

Visit YouTube to see tips and techniques for getting the most out of Shofu's product solutions.

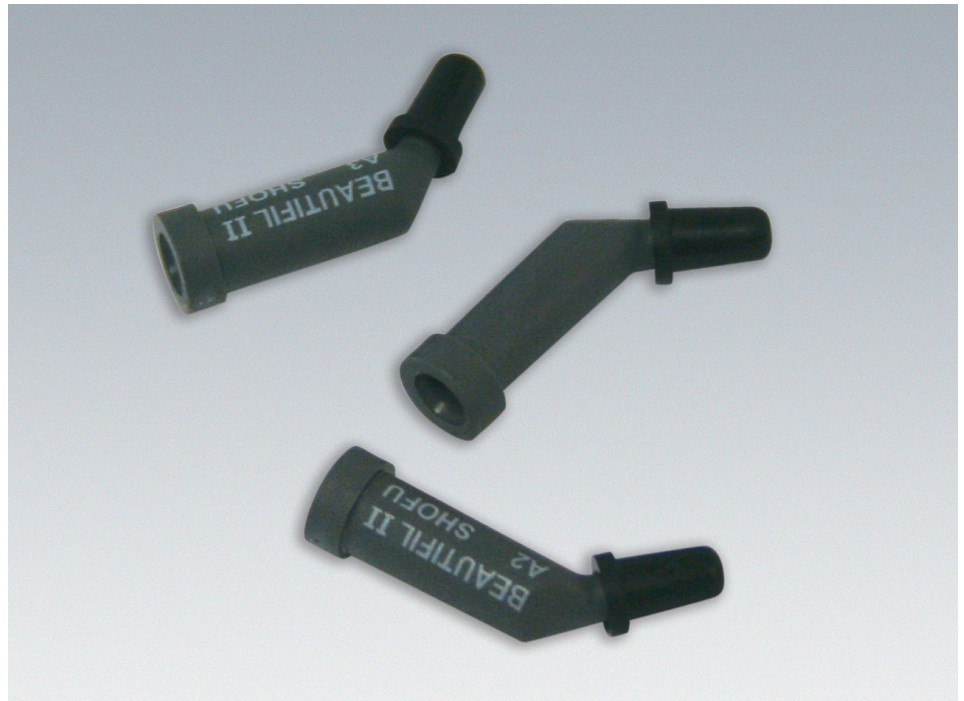

Beautifil II is available in syringes or prefilled tips (pictured above).

naturally fluorescent and radiopaque.

Beautifil II is supplied in prefilled tips or syringes in a wide selection of shades. The syringe has a one touch cap that enhances dispensing and delivery during build-up of restorations.

According to Shofu, the material's thixotropic properties offer easy sculpting without slumping or sticking. Beautifil II is fast and easy to polish, creating an excellent surface luster that is long-lasting.

As Reviewed by Dental Product Shopper

In 2012, 15 *Dental Product Shopper* evaluators evaluated Beautifil II. At the end of the 4-week review period, 12 of

the 15 evaluators said they would definitely or probably recommend Beautifil II to colleagues, and 9 said they would definitely or probably purchase this universal composite in the future. Overall, Beautifil II received a score of 4.1 and earned a "Recommended" rating.

A pediatric dentist with 37 years of experience said, "As a pediatric dentist, the 'holy grail' has always been to develop a fluoride-releasing composite. We may have reached our goal."



SHOFU DENTAL CORPORATION
1225 Stone Drive
San Marcos, CA 92078
FREE Product Information

SN187-0713

Evolve.

GIOMER TECHNOLOGY


taking dentistry to the next level

NEW SHADES!

Beautiful Flow Plus®

Finally, an **Injectable Hybrid Restorative** for All Indications

F00
Zero Flow

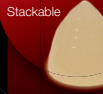

F03
Low Flow



BeautiSealant

Fluoride Releasing Pit & Fissure Sealant System



Beautiful® II

A Nano-Hybrid Composite with Fluoride Release & Recharge



BeautiBond®

One Adhesive: Two Powerful Monomers



NEW!

BeautiCem

Fluoride Releasing Self-Adhesive Resin Cement



Giomer

Scan Here: Acid Neutralization Video



Scan here for more information on Giomer Technology & watch the Acid Neutralization video.



Official Partner



Visit www.shofu.com or call 800.827.4638

Key Features of Giomer Materials

S-PRG filler material clinically:

- Recharges fluoride when treated with fluoridated products
- Decreases acid production of cariogenic bacteria
- Neutralizes acid on contact
- Slows demineralization, while promoting remineralization of enamel
- Demonstrates an anti-plaque effect

