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6. Conditioning and application of adhesive  
Apply dentin adhesive following the manufacturer's instructions for use.

7. Placement  
Dispense adequate amount of BEAUTIFIL II onto a paper pad from the syringe. Apply the dispensed material into the cavity using a suitable instrument and create the shape desired.  
When using a tip, insert the tip into the dispenser to apply the material directly into the cavity. The tip material may also be dispensed onto a paper pad to be applied into the cavity with a suitable instrument.

➤ When storing these materials in a refrigerator, allow them to reach room temperature before use.  
➤ When applying the material directly into the cavity, stabilize the dispenser before and after use.  
➤ Tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.  
➤ Grip the dispenser gently to dispense the material slowly. Excessive pressure may cause Tips to rupture or to eject from the dispenser.  
➤ The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.  
➤ Tightly close the cap after each use.

8. Light-curing  
Light-cure using a dental light-curing unit.  
When the cavity is deep, incremental filling and light-curing not exceeding 2 mm is recommended.  
When the cavity surface is large, light-cure in segments.  
➤ Dental light-curing unit and light-curing time

Light-curing unit	Halogen	LED
Light-curing time	20 sec.	10 sec.

➤ Halogen light-curing unit  
Wavelength: 400-500 nm,  
Light intensity: ≥ 500 mW/cm<sup>2</sup>  
➤ LED light-curing unit  
Wavelength: 440-490 nm,  
Light intensity: ≥ 1000 mW/cm<sup>2</sup>  
➤ Use a dental light-curing unit having specifications equivalent to those described above. When using the one having performance other than these recommendations, follow the instructions of each unit to light-cure BEAUTIFIL II properly.  
➤ Operators must use protective glasses or light-shielding plate to avoid looking directly at the curing light. Operators must also protect patients' eyes from direct exposure to curing light.  
➤ Disinfect the syringe with alcohol after each use to avoid cross-contamination. Then, tightly close the cap. Use caution to avoid including alcohol into the paste.  
➤ Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.  
9. Finishing and polishing  
After light-curing, contour, finish and polish the restoration following the conventional method.

#### NOTE ON USE

- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container.  
(Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)

#### STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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