DIRECTIONS FOR USE

- Pretreatment of the material to be bonded. Perform appropriate pretreatment for each material to be bonded (composite resin, porcelain, metal or natural tooth).
 - Follow the instructions of each pretreatment material.
- Setting of the Needle Tip
 - Follow the instructions below to attach the BEAUTIFIL Needle Tip provided.
 - Pull off the shield cap while turning it slowly. (Fig.A)
 - Attach the needle tip by turning it clockwise until it is securely attached. (Fig.B)
 - Wear clean medical (dental gloves) when replacing the needle tip.

Application
Apply necessary amount of BEAUTIFIL OPAQUER directly to the material to be bonded. BEAUTIFIL OPAQUER may also be extruded onto a paper pad to be applied with a suitable instrument.

- When storing these materials in a refrigerator, allow them to reach room temperature before use.
- The paste dispensed onto the paper pad should be shielded from ambient light by light-shielding cover to avoid unintended polymerization of the material.
- BEAUTIFIL OPAQUER should be applied as thinly as possible (0.5 mm or less)
- Needle tips should be used for only one patient and then discarded at the end of the procedure to avoid cross contamination.

Light-curing
Light-cure using a dental light-curing unit.
When applying thickly, incremental placement and light-curing are required.

- When the cavity surface or the area to be repaired is large, light-cure in segments.
- Dental light-curing unit and light-curing time

Light-curing unit	Halogen	LED
Light-curing time	40 sec.	20 sec.
Wavelength	400-500 nm	440-490 nm
Light Intensity	≥ 500 mW/cm ²	≥ 1000 mW/cm ²

- Use a dental light-curing unit having specifications equivalent to those described above. When using the one having performance other than these recommendations, follow the instructions of each unit to light-cure BEAUTIFIL OPAQUER properly.
- Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light. Operators must also protect patients' eyes from direct exposure to curing light.
- Disinfect the syringe and shield cap with alcohol after each use to avoid cross-contamination. Then, lightly close the cap. Use caution to avoid including alcohol into the paste.
- Disinfect the light-curing unit with alcohol after each use to avoid cross-contamination.

5. Fill direct composite resin and light-cure. Then, contour, finish and polish.
Follow the direct restorative manufacturer's instructions for use.

NOTE ON USE

- Follow the instructions for use of each dental material or instrument to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container.
(Example □ YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

- Store in a dark place at room temperature (1-30 °C /34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.
- Discard the used Needle Tip and store this product in either way of the following.
 - Attach new one. Insert the needle into the small hole on the head of the shield cap. (Fig.C)
 - Replace the shield cap securely. (Fig.D)

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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