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EN

INDICATIONS FOR USE

CERAMAGE OXY-BARRIER is an air inhibition gel. Apply CERAMAGE OXY-BARRIER on the surface of indirect composite restorations before final polymerization to avoid an oxygen-inhibited layer on the composite surface. With high fluidity, it flows easily into micro spaces on the restoration. For easier identification, CERAMAGE OXY-BARRIER is colored in blue.

DIRECTIONS FOR USE

➢ Before final polymerization of composite resin, apply on the composite surface with brush.
➢ After light curing, wash off with water.

NOTE ON USE

1. If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
2. Do not touch the material with bare hands. Avoid contact of this material with skin and eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
3. Use this material in well ventilated area (several appropriate ventilations per hour are recommended).
4. Tightly close the cap immediately after each use.
5. Do not use or store this material near any source of ignition since it is highly flammable.
6. Do not use this product for any purposes other than specifically outlined in these instructions for use.
7. This product is intended for use by dental professionals only.

COMPOSITION

Glycerin, polyethylene glycol, distilled water

STORAGE

Store at room temperature (1-30 °C/ 34-86 °F). Keep away from direct sunlight and any source of ignition.

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Manufacturer
SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA
[EC][REP] SHOFU DENTAL GmbH Am Brühl 17, 40878 Ratingen, Germany
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD. No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL ASIA-PACIFIC PTE. LTD. 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
www.shofu.com

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DIRECTIONS FOR USE

- Before final polymerization of composite resin, apply on the composite surface with brush.
- After light curing, wash off with water.

NOTE ON USE

- If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
- Do not touch the material with bare hands. Avoid contact of this material with skin and eyes. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Use this material in well ventilated area (several appropriate ventilations per hour are recommended).
- Tightly close the cap immediately after each use.
- Do not use or store this material near any source of ignition since it is highly flammable.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- This product is intended for use by dental professionals only.

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