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EN

WARNING

CERAMAGE SPACER contains natural rubber latex. Operators with known allergies to latex must not use this product.

INDICATIONS FOR USE

CERAMAGE SPACER is a spacer for plaster models. Apply the spacer on the plaster die in order to form rubber-like membrane and to ensure space for cement between the completed restoration and prepared tooth. CERAMAGE SPACER also compensates the polymerization shrinkage of resin for bridges and crowns during polymerization.

DIRECTIONS FOR USE

Apply CERAMAGE SPACER evenly on the plaster model except for margin. After drying, it becomes translucent. (Apply CERAMAGE SEP on margin.)

NOTE ON USE

1. In rare cases, natural rubber latex might cause allergic reactions such as itch, rash, urticata, swelling, fever, breathing difficulty, asthma-like symptoms, blood pressure fell and shock to natural rubber. If any allergic reactions occur, immediately discontinue use and seek medical advice.
2. Operators who are often in contact with natural rubber are considered to have a higher risk to develop allergies to latex. Therefore, such operators should use CERAMAGE SPACER carefully.
3. Do not touch the material with bare hands. Use protective gloves to avoid sensitization to this product. In case of accidental contact with skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
4. Use this material in well ventilated area (several appropriate ventilations per hour are recommended).
5. Tightly close the cap immediately after each use.
6. Do not leave this product in the area of high temperature, such as near the heater etc. Avoid direct sunlight.
7. Do not use this product for any purposes other than specifically outlined in these instructions for use.
8. This product is intended for use by dental professionals only.

COMPOSITION

Natural rubber latex

STORAGE

➤ Store at room temperature (1-30 °C/34-86 °F). Keep away from direct sunlight.
Low temperatures, 0 °C/32 °F or lower, might ruin this product by solidifying the material.

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Fles: 7 mL

03:2019-03

Manufacturer
SHOFU INC., 11 Kamitakamatsucho, Fukuoka, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU DENTAL CORPORATION, 1225 Stone Drive, San Marcos, California 92078-4059, USA
[EC REP] SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD., No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL ASIA-PACIFIC PTE. LTD., 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
www.shofu.com