Dura-White Stones

EN Instructions for use

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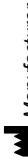
1225 Stone Drive, San Marcos,
California 92078-4059, USA

EN

Made of aluminum oxide grits with ceramic binder, Dura-White Stones are ideal for fine contouring and finishing of enamel, composites and porcelains. They are available in friction grip (FG), contra-angle (CA) and hand piece (HP).

INDICATIONS

Fine contouring and finishing of enamel, composites, and porcelains

PRECAUTIONS

- These abrasives are intended for use by dental professionals only.
- Do not use these abrasives for any purpose other than specifically outlined in INDICATIONS in these instructions for use.

DIRECTIONS FOR USE

- When using intraorally, sterilize the abrasive before each use.
 - Sterilization method: autoclave the abrasive for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
 - For reuse, remove any contaminants, using cleaning liquid or disinfectant and sterilize any of the following methods; autoclave, chemiclave, dry-heat sterilization, disinfectant liquid.
 - Chloride disinfectant liquid (sodium hypochlorite), benzalkonium chloride solution and plasma sterilization must not be used for disinfection as they may cause corrosion or the working part may come off due to degradation to bonding between the head and the shank.
 - Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.
- Mount the abrasive on a dental handpiece (dental electric engine, micro motor, high-speed lathe etc.).
 - Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
 - Perform test run to check safety and smooth operation without run-out prior to use.
 - Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use following the conventional manner.
 - Never exceed the maximum permissible rotating speed.

Maximum permissible rotating speed	30,000 min ⁻¹
------------------------------------	--------------------------

- Apply gentle pressure under water spray to avoid damaging pulp.
 - Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
 - Use protective equipment such as glasses while using.
 - Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- For reuse after using intraorally, remove any contaminants immediately using cleaning liquid or disinfectant and sterilize.

STORAGE

Store these abrasives avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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