SHOFU GlasIonomer Base Cement

EN Instructions for use	ES Instrucciones de uso	RO Instrucțiuni de utilizare
DE Gebrauchsanleitung	IT Istruzioni per l'uso	SV Bruksanvisning
FR Mode d'emploi	RU Инструкция по применению	PT Instruções de uso
NL Gebruiksaanwijzing	PL Instrukcja obsługi	

03:2019-03
71323

<div></div> <div>Manufacturer</div>	<div></div> <div>EC REP</div>
SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan www.shofu.com	SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos, California 92078-4059, USA

EN Dentin Restorative

Glasionomer Base Cement is a radiopaque, glass ionomer base cement for dentin replacement. Glasionomer Base Cement chemically bonds to dentin and enamel.

SHADES
YELLOW, WHITE

INDICATIONS

➤ Base/Liner

➤ Core build-up

PRECAUTIONS

- Do not use this product on patients with known allergies to this material and/or glass polyalkenoate cements.
- Operators with known allergies to this material and/or glass polyalkenoate cements should not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- The liquid and mixture of this product should not come in contact with intraoral soft tissue, skin or eyes. In case of accidental contact with intraoral soft tissue or skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

- Cleaning of tooth surface
 - Thoroughly clean the tooth surface to remove plaque.
- Cavity preparation
 - Prepare the cavity following the conventional method.
- Dispensing of Powder and Liquid (standard powder/liquid ratio: 2.6 g/1.0 g)
 - Dispense one level scoop of Powder with the measuring scoop provided. Then, dispense one drop of Liquid separately.
 - Always use the measuring scoop provided and level off the powder for accurate dispensing.
 - Invert the liquid bottle prior to use to dispense drops of bubble-free liquid.
 - Remove any liquid plugging the opening of the nozzle with wet gauze etc. for accurate measuring.
 - Tightly close the bottle caps (powder and liquid) immediately after each use.
 - Use extra caution to keep the powder away from any moisture.
- Mixing
 - Divide dispensed powder into 2 equal portions; introduce the first half to the dispensed liquid and mix for about 15-30 sec. with spatula. Then, add the remaining half and mix until it becomes homogeneous paste.

Mixing time (23±1 °C/73.4±1.8 °F)	Within 60"	
Working time (23 °C/73.4 °F)	2' 30"	From start of mixing
Setting time (37 °C/98.6 °F)	2' 30"	From end of mixing

- Do not add extra liquid while mixing to prevent degradation of the performance of this material.
- Base/Lining/Core build-up
 - Base, line or build up core following the conventional method.
 - When the cement mixture loses luster, apply cocoa butter to the restored area to protect from saliva contamination.
 - The cement mixture turns whitish when it is contaminated with moisture immediately after application. Control moisture completely. After filling, apply varnish etc to protect from moisture contamination.
 - When restored area cannot be protected from moisture completely, the surface of the set material might become whitish after contouring. Thus, apply the material oversized by about 0.5mm.
 - Do not desiccate the material to prevent micro-cracks.

NOTE ON USE

- When taking an impression immediately after filling this material, apply vaseline to the surface to prevent surface roughness of the stone caused by acidity.
- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package and container.

(Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)

STORAGE

Store at room temperature (1-30 °C/34-86 °F). Avoid high humidity. Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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