<b>EN</b>	<span>READ CAREFULLY BEFORE USE</span>
-----------	----------------------------------------

### ENHANCED DIRECT RESTORATIVE

#### PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS

- Do not use this product on patients with known allergies to this material and/or glass polyalkenoate cements.
- Operators with known allergies to this material and/or glass polyalkenoate cements must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator while using this product, immediately discontinue use and seek medical advice.
- Avoid contact with intraoral soft tissue, skin or eyes. In case of accidental contact with intraoral soft tissue or skin, immediately blot with alcohol moistened cotton ball and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- This product is intended for use by dental professionals only.

#### INDICATIONS

- Restoration of deciduous teeth
- Non-load bearing Class I and Class II restorations in permanent teeth
- Restorations of Class III and Class V cavities and wedge shaped defects
- Core build-up and base/liner

#### DIRECTIONS FOR USE

- Shade selection**

Select one shade optimal for the patient.
- Cavity preparation**

Prepare the cavity following the conventional method.
- Isolation**

Isolate with rubber dam etc.
- Pulp capping**

In order to prevent pulp irritation, apply calcium hydroxide to the area around the pulp.
- Dispensing of powder and liquid (standard powder/liquid ratio: 2.6 g/1.0 mL)**

Dispense one level scoop of powder onto a paper pad with the measuring scoop provided. Then, dispense one drop of liquid separately.
- Mixing**

Divide dispensed powder into 2 equal portions; introduce the first half to the dispensed liquid and mix for 15-30 sec. with spatula. Then, add the remaining half and mix until it becomes homogeneous paste.

Mixing temperature	23 ± 1°C/73.4 ±1.8 °F
Mixing time	40"
Working time (23°C/73.4°F)	2'00"
Setting time (37°C/98.6°F)	2'30"

- Filling and moisture control**

Rinse the prepared tooth with water and dry. Then fill the mixture using an instrument and create the intended shape. When the cement mixture loses luster, apply cocoa butter to the restored area to protect from moisture contamination.
- Finishing/polishing**

Finish and polish on the next day or later following the conventional method.

#### NOTE ON USE

- Always use the measuring scoop provided. Level off the powder with the built-in leveller for accurate dosage.
- Invert the liquid bottle prior to use to dispense drops of bubble-free liquid.
- Remove any liquid plugging the opening of the nozzle with wet gauze etc. for accurate measuring.
- Do not add extra liquid while mixing as this will affect material properties.
- Do not mix powder or liquid with components of other glass polyalkenoate cements.
- Temperatures higher than 23°C will shorten the working time, and those lower than 23°C will extend it.
- Tightly close the caps (powder, liquid and cocoa butter) immediately after each use. Use extra caution to keep the powder away from any moisture.
- In the case where pulp is exposed or the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.
- When using a strip, apply some cocoa butter before placing it.
- Cocoa butter might be separated; however, it does not affect the quality of the product.
- Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example ☞ YYYY-MM-DD→Year-Month-Date of the expiration date)

#### STORAGE

Store at room temperature (1-30°C/34-86 °F). Avoid high temperature and high humidity. Keep away from direct sunlight and any source of ignition.

#### COMPOSITION

Powder: Fluoroaluminosilicate glass, pigments and fluorescent material
Liquid: Acrylic acid tri-carboxylic acid co-polymer solution, tartaric acid
Cocoa butter: Cocoa butter, white vaseline

#### SHADE

A2, A3, A3.5, B2

***CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.***

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