Hybrid Points Kit

03:2019-04
71402

MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

0901, 0903, 0905, 0907, 0909	450,000 min⁻¹
0911, 0913, 0915, 0917, 0919, 0921, 0923, 0925, 0927, 0929, 0931, 0933, 0935, 0937, 0939, 0941, 0943	300,000 min⁻¹

DE MAXIMAL ZULÄSSIGE DREHZAHLEN (Tabelle A)
FR VITESSE DE ROTATION MAXIMUM AUTORISEE (Tableau A)
NL MAXIMAAL TOEGESTANE ROTATIESNELHEID (Tabel A)
ES VELOCIDAD DE ROTACIÓN MÁXIMA PERMITIDA (Tabla A)
IT MASSIMA VELOCITÀ DI ROTAZIONE CONSENTITA (Tabella A)
SV HÖGSTA TILLÅTNA ROTATIONSHASTIGHET (Tabell A)
PT VELOCIDADE MÁXIMA DE ROTAÇÃO ADMISSÍVEL (Tabela A)

EN

READ CAREFULLY BEFORE USE

SHOFU Hybrid Points, diamond-coated abrasives are used for cavity, crown and bridge preparation.

NOTES

- Never exceed the maximum permissible rotation speeds described in Table A.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation and dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Use intermittently with gentle touch and water spray.
- Avoid any use at acute angles or with excessive pressure to thin and long abrasives.
- Use local dust extractor and protective equipment while using.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

STERILIZATION

- Perform autoclave under the condition described in Table B.
- When using intraorally, sterilize before each use.
- For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.
- Do not use chlorine disinfectant liquid.
- Follow the instructions for use of washer disinfectant and sterilizer, respectively.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

For the details about **REPROCESSING** procedures, visit www.shofu.de.

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