SHOFU OneGloss Mini-point

- EN** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- SV** Bruksanvisning
- DE** Gebrauchsanleitung
- NL** Gebruiksaanwijzing
- IT** Istruzioni per l'uso
- PT** Instruções de uso

03:2019-03
300030284

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684


Mini-point PN 0186

Maximum permissible speed
15,000 min⁻¹

For
Fissure,
Cervical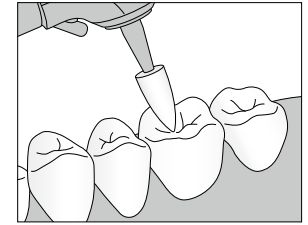

EN

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

OneGloss is a one-step finisher and polisher that provides the ideal finish for composite restorations, simply by altering the contact pressure as you go from finishing to polishing.

INDICATIONS FOR USE

- > Finishing and polishing of composite restoration
- > Removal of excess resin or cement on tooth surface
- > Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- > Polishing of enamel after tooth scaling
- > Removal of stain from tooth surface

PRECAUTIONS AND WARNINGS

1. Do not use damaged or defective instrument, i.e. deformation, dirt, corrosion, exposure of shank due to repeated use.
2. Use protective mask and eyewear when operating handpieces.
3. Do not use this product for any purposes other than specifically outlined in these instructions for use.
4. The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
2. Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Mini-point	Fissure, cervical	1.0 N	0.3 N

- > Use at rotation speed of between 3,000 min⁻¹ to 10,000 min⁻¹ with intermittent water spray. Avoid exceeding maximum permissible speed, 15,000 min⁻¹.
 - > Use saliva ejector or suction to eliminate moisture and debris during the procedure.
3. After using OneGloss Mini-point, remove any contaminants immediately, disinfect and sterilize.

STERILIZATION AND STORAGE

OneGloss Mini-point should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination.

When using intraorally, sterilize before use. Remove any contaminants using suitable cleaning liquid or disinfectant and perform autoclave.

Product name	Autoclave
OneGloss Mini-point	134 °C/273 °F - 3'00" 121 °C/250 °F - 30'00"

Follow manufacturer's instructions for use of cleaning liquid, disinfectant or sterilizer. Chloride disinfectant should be avoided.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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