- EN** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- RU** Инструкция по применению
- RO** Instrucțiuni de utilizare
- EL** Οδηγίες χρήσης
- DE** Gebrauchsanleitung
- NL** Gebruiksaanwijzing
- IT** Istruzioni per l'uso
- PL** Instrukcja obsługi
- SV** Bruksanvisning
- PT** Instruções de uso

03:2019-03
300030290

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com






EC REP

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

CE 0123

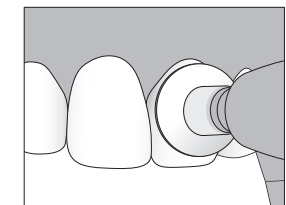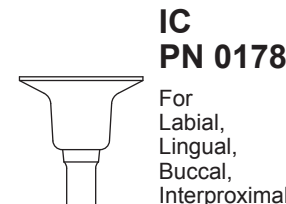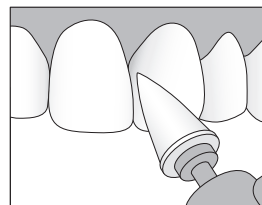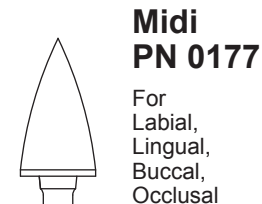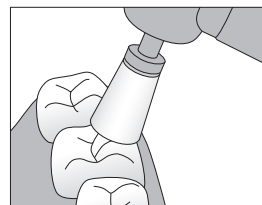Do not re-use
OneGloss PS

Maximum permissible speed

5,000 min⁻¹

EN

READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTION

OneGloss PS is a disposable one-step finisher and polisher with plastic shank that provides the ideal finish for composite restorations, simply by altering the contact pressure as you go from finishing to polishing.

INDICATIONS FOR USE

- Finishing and polishing of composite restoration
- Removal of excess resin or cement on tooth surface
- Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- Polishing of enamel after tooth scaling
- Removal of stain from tooth surface

PRECAUTIONS AND WARNINGS

1. OneGloss PS polishers are designed for single use only. Discard the polisher after use in accordance with the disposal protocol of your healthcare facility.
2. Do not use damaged or defective polishers, i.e. deformation, dirt.
3. Use protective mask and eyewear when operating handpieces.
4. Do not use this product for any purposes other than specifically outlined in these instructions for use.
5. The product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
2. Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Cup	Cusp, incisal, occlusal	1.0 N	0.3 N
Midi	Occlusal, labial, lingual, buccal		
IC	Labial, lingual, buccal, interproximal		

- Use at rotation speed of between 3,000 min⁻¹ to 5,000 min⁻¹ with intermittent water spray. Avoid exceeding maximum permissible speed, 5,000 min⁻¹.
 - Use saliva ejector or suction to eliminate moisture and debris during the procedure.
3. Dispose OneGloss PS polishers after each use.

STORAGE

OneGloss PS should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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