

## REQUIRED SPEEDS

	REQUIRED SPEED (RPM)	WATER SPRAY
AcryPoint	5,000 - 15,000	no
Brownie, Greenie, Supergreenie	5,000 - 30,000	yes
Brownie, Greenie, Supergreenie unmounted; Supergreenie PC2	5,000 - 20,000	yes
Brownie, Greenie, Supergreenie Floppie & Cup (FG)	5,000 - 15,000	no
CeraMaster Coarse & CeraMaster	10,000 - 20,000	no
Ceramisté	10,000 - 20,000 (HP/CA) 10,000 - 15,000 (FG)	yes
Ceramisté Soft	5,000 - 20,000	yes
Ceramisté Soft Unmounted	5,000 - 15,000	yes
Chipless Wheels	5,000 - 20,000	no
CompoMaster Coarse & CompoMaster	5,000 - 20,000 (HP/CA) 5,000 - 15,000 (Wheel)	no
CompoSite & CompoSite Fine	10,000 - 15,000 (FG) 10,000 - 20,000 (CA)	yes
CoreMaster Coarse & CoreMaster	10,000 - 20,000	no
Dura-Green, Fine, DIA & Dura-White	5,000 - 30,000	no
Dura-Green unmounted wheels	5,000 - 30,000	no
Hardie Black & Hardie Brown	5,000 - 30,000	no
Hardie Black & Hardie Brown Floppie	5,000 - 15,000	no
Lab Series Stones	5,000 - 30,000	no
Lab Series Stones Unmounted	5,000 - 30,000	no
OneGloss	3,000 - 15,000	yes
OneGloss PS	3,000 - 5,000	yes
SuperBuff	8,000 - 15,000	moisten before using to generate paste
Super-Snap & Super-Snap X-Treme	10,000 - 15,000	no
Super-Snap Buff Disk	5,000 - 10,000	no
ZiLMaster	5,000 - 20,000 (HP/CA) 5,000 - 15,000 (Wheel)	no

	REQUIRED SPEED (RPM)	WATER SPRAY
<b>SHOFU ROBOT FG POINT</b>		
0897, 0893, 0897, 0899, 899V	0 - 120,000	yes
0822, 0826, 0885, 0886, 0890, 0892, 0894, 0895, 0898, 895C, 892X, 895X, 898X, 826V, 893V	0 - 160,000	yes
0790, 0792, 0793, 0800, 0803, 0810, 0814, 0818, 0820, 0824, 0825, 0830, 0831, 0833, 0834, 0835, 0836, 0837, 0838, 0839, 0840, 0844, 0846, 0848, 0850, 0852, 0858, 0866, 0870, 0876, 0878, 0880, 0881, 0883, 0888, 0891, 0896, 817C, 835C, 840C, 845C, 883C, 835X, 840X, 845X, 850X, 876X, 883X, 820F, 835F, 840F, 846F, 847F, 848F, 850F, 883F, 790V, 792V, 793V, 808V, 825V, 833V, 836V, 839V, 840V, 841V, 843V, 880V, 883V	0 - 300,000	yes
0789, 0791, 0801, 0805, 0807, 0812, 0813, 0816, 0832, 0842, 0855, 0856, 0859, 0860, 0861, 0864, 0865, 0868, 0872, 0874, 0875, 0879, 832X, 832F, 842F, 875F, 789V, 791V, 874V	0 - 450,000	yes
<b>SHOFU ROBOT FG POINT MINIMALLY INVASIVE</b>		
0944, 0945, 0946, 0947, 0948, 0949	0 - 300,000	yes
<b>SHOFU ROBOT CARBIDE BURS</b>		
0602, 0603, 0605, 0606, 0609, 0610, 0613, 0614, 0616, 0617, 0620, 0623, 0626, 0630, 0631, 0634, 0667	0 - 50,000	no
0719, 0727, 0761-3, 0762-3, 0763-3, 0770-3, 0771-3, 0772-3	0 - 300,000	yes
0703, 0704, 0709, 0710, 0717, 0718, 0736, 0741, 0742, 0743, 0764-3, 0765-3, 0766-3, 0767-3, 0768-3, 0769-3,	0 - 450,000	yes
<b>GINGI CURETTAGE</b>		
C063	0 - 450,000	yes
C061, C062, C081, C082, C083, C084, C085	0 - 300,000	yes
C101, C102, C103	0 - 160,000	yes
<b>T&amp;F HYBRID POINTS</b>		
0925, 0927, 0929, 0931, 0933, 0935, 0937, 0939	0 - 450,000	yes
0941, 0943	0 - 300,000	yes
<b>HYBRID POINTS</b>		
0901, 0903, 0905, 0907, 0909	0 - 450,000	yes
0911, 0913, 0915, 0917, 0919, 0921, 0923	0 - 300,000	yes
<b>TPE DIAMONDS</b>		
684C, 686C, 684F, 686F, 682S, 684S	0 - 450,000	yes
680C, 682C, 680F, 682F	0 - 300,000	yes