PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS
 1. Do not use this product on patients with known allergies to this material, acetone and/or methacrylate monomer.
 2. Operators with known allergies to acetone, methacrylate monomer must not use this product.
 3. If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
 4. Use protective medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material come in contact with the gloves, immediately remove them and wash hands with plenty of water.
 5. In case of contact with eyes, flush immediately with plenty of water and seek medical advice.
 6. Use in well ventilated area to avoid inhalation of acetone. If inhalation causes nausea or illness, take rest and keep warm in fresh air.
 7. This product is intended for use by dental professionals only.

APPLICATION
 Adhesive cementation of Alumina or Zirconia restorations, porcelain or indirect composite resin laminate veneers, crowns, bridges, inlay, onlay and metal reinforced indirect restorations. Recommended for cementation of various types of posts and core materials


DIRECTIONS FOR USE
I Pretreatment of cavity or abutment tooth
 1. Remove temporary seal and cement completely and clean cavity or abutment tooth using the conventional method.
 2. Pulp protection: In case where cavity is close to pulp, pulp capping with calcium hydroxide is indicated. Isolate using rubber dam to prevent moisture contamination.
 3. In case of enamel or dentin adhesive surface on abutment tooth: Dispense one drop each of Primer A and B on a dish and mix, then apply the primer on the abutment, leave it undisturbed for 20 seconds and air dry.
Note: when cementing a porcelain laminate, treat enamel surface with a phosphoric acid etchant (phosphoric acid concentration 30-40%) before primer treatment.
 > BeautiBond Universal also can be used for surface treatment of the abutment. Refer to the instructions for use of BeautiBond Universal in use.

4. In case of metal adhesive surface on abutment tooth: Apply a uniform layer of M.L. Primer / SHOFU Universal Primer on the adhesive surface with a microtip and leave it undisturbed for 10 seconds.
 5. In case of composite resin adhesive surface on abutment tooth: Apply Porcelain Primer on the adhesive surface using a microtip and leave undisturbed for 10 seconds.
 6. In case of Alumina or Zirconia adhesive surface on abutment tooth: Apply AZ Primer / SHOFU Universal Primer on the alumina or zirconia surface using a microtip and leave undisturbed for 10 seconds.
II Pretreatment of restorations
 1. Sandblast the adhesive surface of the restoration with alumina sand of 50-100 µm at each recommended pressure gauge. Perform ultrasonic cleaning in water and air dry.
 2. Dispense an adequate amount of specified primer (M.L. Primer, Porcelain Primer, AZ Primer, SHOFU Universal Primer) on a dish and apply a uniform layer on the entire adhesive surface with microtip. Leave undisturbed for 10 seconds.

Recommended pressure and primer for each restorations

Restorations	Pressure	Primer
Metal	about 0.3-0.5 MPa (about 3-5 kgf/cm ²)	M.L. Primer SHOFU Universal Primer
Porcelain, Indirect Composite resin	about 0.1-0.2 MPa (about 1-2 kgf/cm ²)	Porcelain Primer
Alumina or Zirconia ceramics	about 0.2-0.3 MPa (about 2-3 kgf/cm ²)	AZ Primer SHOFU Universal Primer

> Refer to the manufacturer's instructions for use to select optimal pressure.
 > BeautiBond Universal also can be used for surface treatment of the restorations. Refer to the instructions for use of BeautiBond Universal in use.
III Dispensing of ResiCem Paste
 1. Preparation of syringe and accessories
 Turn the cap a 1/4 turn counterclockwise to fit the small projection on the cap into the notch of the syringe. Remove cap by holding the foot end of the cap and pulling downward.
 Extrude some paste to check if the Paste A and B get dispensed evenly. Check this before each use.
 Attach the mixer tip supplied to the notch at the end of the syringe and turn 90° in a clockwise direction.

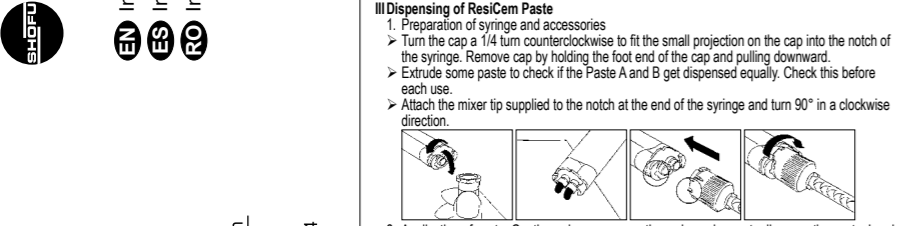2. Application of paste: Gently apply pressure on the syringe plunger to dispense the paste. Load paste onto the adhesive surface of the pretreated restoration.
 3. Seating of restoration and removal of excess paste: Seat the restoration on the abutment tooth or cavity and press. Remove excess paste thoroughly using a microtip etc. Light cure using dental light-curing unit. Halogen: 20 seconds, LED (high power type over 1000 mW/cm²): 10 seconds. The area where light cannot reach must be chemically cured. Working time (at 23°C / 73°F) is about 3 minutes. Chemical curing starts in 30 seconds (with primer) or 90 seconds (without primer) after restorations were placed. Standard chemical curing time in the mouth is about 4 minutes.
 4. Finishing and polishing: Perform marginal finishing and polishing with occlusal adjustment, following the conventional method.

NOTE ON USE
 > Store at refrigerator (1-10°C/34-50°F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of heat. Do not freeze.
 > Allow the materials to reach room temperature before use.
 > Do not use eugenol-containing material and Hydrogen peroxide since they might retard the curing process and weaken the bond strength.
 > Use saliva ejector or a vacuum during treatment.
 > Close the cap immediately after dispensing each primer and use dispensed material promptly.
 > Use caution to prevent the adhesive surface from being contaminated by saliva or blood, etc. before bonding. If the surface is contaminated by saliva or blood, thoroughly clean the surface with an alcohol moistened cotton ball and dry. Then, reapply the material.
 > Close the syringe with the original cap or store with the used mixer tip after use. Note that the used mixer tip must be replaced with a new one before the next application. In case of using original cap, remove any residual paste before placing the cap.
 > Take extra caution to avoid polymerization of paste while syringe is uncapped.
 > Brushes (microtips) and dishes are for single use only.
 > Primer must be applied only on tooth surface. (Do not apply on restoration.) Once paste contacts with primer treated surface, the setting reaction is accelerated.
 > Discard the hardened paste at the tip of the used syringe after storage for a long period of time.
 > Light-cure the margin sufficiently. (For placing a restoration made of CAD/CAM resin block etc., sufficient light-curing on the margin ensures stable cementation.)
 > Operators must use protective glasses or light shielding plate to avoid direct exposure to the curing light. Operators must also protect patients' eyes from the direct exposure to the curing light.
 > Disinfect the light-curing unit with alcohol after each use to avoid cross contamination.
 > Do not use this product for any purposes other than specifically outlined in these instructions for use.
 > Use this product within the expiration date indicated on the package.
 (Example: YYY-MM-DD → Year-Month-Date of the expiration date)

COMPOSITION
 Paste A: UDMA, TEGDMA, Fluoroaluminosilicatglass, Initiator
 Paste B: UDMA, TEGDMA, Carboxylic acid monomer, 4-AET, 2-HEMA, Fluoroaluminosilicatglass, Initiator
 Primer A: Water, Acetone, Initiator
 Primer B: 2-HEMA, 4-AET, Carboxylic acid monomer, Acetone

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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