SHOFU SOLIDEX
SOLI BOND
METAL PHOTO PRIMER

EN Instructions for use
DE Gebrauchsanleitung
FR Mode d'emploi
NL Gebruiksaanwijzing
ES Instrucciones de uso
IT Istruzioni per l'uso

03:2019-03
71601

SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC REP
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany
SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos, California
92078-4059, USA
SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II,
Singapore 117684

EN READ CAREFULLY BEFORE USE

SOLIDEX METAL PHOTO PRIMER

Due to the identical light curing properties of SOLIDEX METAL PHOTO PRIMER and the specific fillers of the paste opaque, an excellent adhesion is guaranteed between metal and composite even in the undercut areas of the retention.

Dispense 1-2 drops on a dish and apply a thin layer onto the surface to be veneered (use SHOFU UNI BRUSH #4). Allow the primer to dry for 30-60 seconds and then apply the first layer of opaque.

SOLIDEX SOLI BOND

SOLIDEX SOLI BOND is a silane coupling agent which can be used with silane bonding systems, i.e. Silicoater, Silicoater MD (Heraeus Kulzer) or Rocatec (3M ESPE). These systems can be used with SOLIDEX without problems.

Do not apply METAL PHOTO PRIMER for these procedures.

Consider the instructions for use from the manufacturer of the silane bonding system and apply Solibond instead of the activator-liquid as the final step.

(Notes)

- When storing these materials in a refrigerator, allow them to reach room temperature before use.
- Tightly close the cap after each use.

PRECAUTIONS AND WARNINGS

1. Do not use this product on patients with known allergies to this material.
2. Operators with known allergies to this material and/or methacrylate monomer must not use this product.
3. If any inflammation or other allergic reactions occur on operator, immediately discontinue use and seek medical advice.
4. Since SOLI BOND and METAL PHOTO PRIMER are inflammable, avoid any source of fire.
5. Use this material in well ventilated areas (several appropriate ventilations per hour are recommended).
6. Follow the instructions for use of each dental material, instrument or equipment used in conjunction with this product.
7. Do not use this product for any purposes other than specifically outlined in these instructions for use.
8. Use this product within the expiration date indicated on the package.
(Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)
9. This product is intended for use by dental professionals only.

COMPOSITION

- SOLIDEX METAL PHOTO PRIMER: Acetone, 4-AET and others
- SOLIDEX SOLI BOND: Dehydrated ethanol, Silane coupling agent

STORAGE

Store in a dark place at room temperature (1-30 °C/34-86 °F). Avoid high temperatures and any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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