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VINTAGE Art Stain Liquid

Please refer to the note on use for optimal effect

English Mixing Liquid

Ingredients

Propylene glycol

Indication

Mixing Liquid for dilution or mixing of VINTAGE Art Stain and Glazing Paste

Note on Use

Do not use with other products besides VINTAGE Art Stain and Glazing Paste.

Caution

- Do not use or keep near fire as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperature or direct sunlight.
- Keep it tightly closed if not in use.
- Do not use for purpose other than that described here.
- This product is for use by dental professional only.

Important

- Do not touch with hand. Pay attention not to contact skin or eyes.
- In case of skin contact, wipe off with cotton wool soaked in alcohol and rinse immediately with running water.
- In case of eye contact, rinse immediately with plenty amounts of water and contact a doctor.
- If eruption or inflammation occurs, stop using this product and contact a doctor.

Storage

Keep away from heat, direct sunlight or flame. Seal the cap tightly and keep at room temperature.

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Vintage Art 染色剂 50mL

Manufacturer
SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, California 92078-4059, USA
EC REP SHOFU DENTAL GmbH Am Brill 17, 40878 Ratingen, Germany
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD., No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China
SHOFU DENTAL ASIA-PACIFIC PTE. LTD., 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684
www.shofu.com