ZILMaster

EN Instructions for use

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01:2014-11
XXXX000000

EN

ZILMaster is a diamond impregnated three-step polishing system designed for achieving high surface gloss on Zirconia and Lithium disilicate restorations; this three step system smoothens the restoration and removes surface scratches during contouring, and provides high gloss polish. ZILMaster is available in a variety of shapes in HP, CA and FG for both extra and intra-oral application.

SHAPES

HP: Bullet Mini-Point, KN7, WH6 and Unmounted Wheel

CA: Bullet, Knife, Cup and Mini-Point

FG: Mini-Point

INDICATIONS

Coarse: Finishing & pre-polishing of Zirconia or Lithium disilicate restorations

Medium: Polishing of Zirconia or Lithium disilicate restorations

Fine: Final polishing of Zirconia or Lithium disilicate restorations

PRECAUTIONS

- This product is intended for use by dental professionals only.
- Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.

DIRECTIONS FOR USE

- When using intraorally, sterilize this product before each use.
 - Sterilization method: autoclave this product for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
 - Remove any contaminants, using cleaning liquid or disinfectant and sterilize with any of the following methods; autoclave, chemiclave, or disinfectant liquid. (Do not perform dry heat sterilization.)
 - Do not immerse this product into chlorine disinfectants for a prolonged period of time.
 - Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.
- Mount this product on a dental handpiece (dental electric engine, micro motor, high-speed lathe etc.).
 - Insert the shank completely into the chuck of the handpiece, following the manufacturer’s instructions.
 - Perform test run to check safety and smooth operation without run-out prior to use.
 - Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use following the conventional manner.
 - Never exceed the maximum permissible rotating speed (20,000min⁻¹).
 - Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
 - Use protective equipment such as glasses while using.
 - Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
 - Do not apply excessive pressure or exceed the recommended speed (RPM), to avoid generating frictional heat that may damage the tooth surface or pulp.
- For reuse after using them intraorally, remove any contaminants immediately using cleaning liquid or disinfectant and sterilize.

STORAGE

- Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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FORMEN

HP: Bullet Mini-Point, KN7, WH6 und Unmounted Wheel

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FORMES

HP(PM): Bullet Mini-Point, KN7, WH6 and Unmounted Wheel

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VORMEN

HP(Handstuk): Bullet Mini-Point, KN7, WH6 en Unmounted Wheel

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FG: Mini-Point

INDICATIONS

Coarse: Finishing & pre-polishing of Zirconia or Lithium disilicate restorations
Medium: Polishing of Zirconia or Lithium disilicate restorations
Fine: Final polishing of Zirconia or Lithium disilicate restorations

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DIRECTIONS FOR USE

- When using intraorally, sterilize this product before each use.
 - Sterilization method: autoclave this product for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
 - Remove any contaminants, using cleaning liquid or disinfectant and sterilize with any of the following methods; autoclave, chemiclave, or disinfectant liquid. (Do not perform dry heat sterilization.)
 - Do not immerse this product into chlorine disinfectants for a prolonged period of time.
 - Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.
- Mount this product on a dental handpiece (dental electric engine, micro motor, high-speed lathe etc.).
 - Insert the shank completely into the chuck of the handpiece, following the manufacturer’s instructions.
 - Perform test run to check safety and smooth operation without run-out prior to use.
 - Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use following the conventional manner.
 - Never exceed the maximum permissible rotating speed (20,000min-1).
 - Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
 - Use protective equipment such as glasses while using.
 - Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
 - Do not apply excessive pressure or exceed the recommended speed (RPM), to avoid generating frictional heat that may damage the tooth surface or pulp.
- For reuse after using them intraorally, remove any contaminants immediately using cleaning liquid or disinfectant and sterilize.

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